

MIGMA 200

The compound for precise impressions, instantly ready.



1 Chose the mixing time on the scale.



2 Dose the alginate as suggested by the producer and fill into the mixing bowl.



3 Add the correct amount of water either with the producer's measuring cup or with the optional available water flask.



4 Close the lid of the mixing bowl with a twist to the right until the click.



5 Shake the bowl once or twice.



6 Put the mixing bowl into the machine and close the lid of the machine.



7 Press START. The machine stops automatically when the chosen time is up.



8 With the spatula the alginate can now easily be removed from the bowl.



9 Within seconds a homogenous, bubble-free impression compound.

MIKRONA TECHNOLOGIE AG

Wigartestrasse 8, CH-8957 Spreitenbach, T +41 56 418 45 45, F +41 56 418 45 00, swiss@mikrona.com

www.mikrona.com

MIKRONA
The Swiss Dental Technology