

Cleaning and disinfecting Mikrona treatment units

Cleaning upholstery:

With FD 360 - Cleaning and care for vinyl upholstery of Dürr Dental AG, DE-74302 Bietigheim
Cleaning and care fluid for the deep-cleaning and care for vinyl upholstery surfaces. Intensive cleaning effect-removes stains of all kind. Dirt rims and textile discolouring, e.g. from unsealed colours in clothing. Special sponges for to remove hard dirt. Set with 500-ml FD-360 Spray bottle and 10 special sponges. Selling by the qualified specialized trade with Dürr products. (*Mikrona order number: 320.3150.01*)

Application:

- Once a week or as required
- Spray the Odourless, undiluted cleaning fluid onto the be cleaned surfaces
- Wipe clean with a cloth
- For intensive cleaning rub FD 360 on with the special sponge
- Subsequently wipe off the excess FD 360 with a dry cloth

Product:

Designation	max. concentration	Manufacturer
FD 360	pure	Dürr Dental, DE-Bietigheim
Ferrari Clean	pure	Ferrari-Textiles, FR-La Tour du Pin

Disinfection upholstery:

Do not use in any case disinfectants products or any other cleaning solvents for cleaning the upholstery!

These products do not have a sufficient cleaning efficiency and should economically, it means exclusively for the purpose of the disinfection be used. The contained substances extract the softeners, which can lead to embitterment, discolorations and cracking the upholstery.

After disinfection the upholstery, use a dry cloth for drying so that no disinfection liquid can infiltrate into the upholstery.

With that in the following specified disinfectants liquid the impairment of the upholstery is small, if the indicated quantity okay and the application are correctly

Product:

Designation	max. concentration	Manufacturer
Bacoban	5,0%	Adexano, DE-Neunkirchen
PlastiSept*	pure	Alpro, DE-St. Georgen
PlastiSept-wipes*	pure	Alpro, DE-St. Georgen
Acrylan	pure	Anticeptica, DE-Pulheim
Descocid	2,0%	Anticeptica, DE-Pulheim
Bacillocid Spezial	2,0%	Bode Chemie, DE-Hamburg
Bacillol 25	pure	Bode Chemie, DE-Hamburg
Kohrsolin FF	2,0%	Bode Chemie, DE-Hamburg
Microbac forte	2,0%	Bode Chemie, DE-Hamburg
Microbac extra	2,5%	Bode Chemie, DE-Hamburg
Hexaquart forte	2,0%	Braun Medical, DE-Melsungen
Hexaquart Plus	1,5%	Braun Medical, DE-Melsungen
Meliseptol Foam fresh	pure	Braun Medical, DE-Melsungen
Melsitt	3,0%	Braun Medical, DE-Melsungen
Quatohex	5,0%	Braun Medical, DE-Melsungen
SprayIn	pure	Deppe, DE-Kempen
SprayOff	pure	Deppe, DE-Kempen
FD 312*	1,0%	Dürr Dental, DE-Bietigheim
FD 312 wet wipes*	pure	Dürr Dental, DE-Bietigheim
FD 366 sensitiv	pure	Dürr Dental, DE-Bietigheim
FD 366 sensitiv wipes	pure	Dürr Dental, DE-Bietigheim
Incidin Plus	5,0%	Henkel, DE-Düsseldorf
Incidin Extra N	5,0%	Henkel, DE-Düsseldorf
Incidur	3,0%	Henkel, DE-Düsseldorf
Kanitop-AF*	pure	Kaniedenta, DE-Herford
Köhler Sprühdesinfektion	pure	Köhler, DE-Neckarsulm
Medichem Flächendesinfektion	0,5%	Medichem, DE-Rendsburg
Medichem Desinfektionschaum	pure	Medichem, DE-Rendsburg
Dodenal neu	0,5%	Merck, AT-Wien
Pursept UniSprint Wipes	pure	Merz, DE-Frankfurt a.M.
Biosanitizer-S wipes*	pure	Saniswiss, CH-Genève
Biguanid Fläche	2,0%	Schumacher, DE-Melsungen
Cleanisept wipes	pure	Schumacher, DE-Melsungen
Cleanisept	2,0%	Schumacher, DE-Melsungen
Mikrozid sensitive Liquid*	pure	Schülke&Mayr, DE-Norderstedt
Mikrozid sensitive-wipes*	pure	Schülke&Mayr, DE-Norderstedt
Quartamon Med	7,5%	Schülke&Mayr, DE-Norderstedt
Sagroplus forte	0,5%	Schülke&Mayr, DE-Norderstedt
TPH 5225	0,5%	Schülke&Mayr, DE-Norderstedt
Terralin	0,5%	Schülke&Mayr, DE-Norderstedt
Unisepta Foam-wipes*	Rein	Unident, CH-Genève

***alcohol-free disinfectant**